

CLEANING AND MAINTENANCE FOR YOUR DEARSON 8000 SYSTEM

HAND PIECES

- Start the machine at low RPM
- Attach hand piece
- Clean the diamond burr using Dearson brush and Dearson approved cleaner
- While still running attach the grease gun into the nipple and pump 3 times using Haynes Lubrifilm
- Switch off machine, remove handpiece and dry using a soft cloth
- Store in your box provided by Dearson



DRIVE CABLE

- Clean the outer sheath with a damp cloth
- Dry thoroughly with a soft dry cloth
- Once a week measure the distance between the inner and outer cable. This should be 25mm and adjust using the allen key provided by Dearson
- Once a month unscrew and slide down the outer cable from the inner cable. Wipe off the excess grease with paper towel and re grease with Foredom grease
- Store in your box provided by Dearson



MOTOR

- Clean the motor with a damp cloth
- Dry thoroughly with a soft dry cloth
- Pouches can be washed using a domestic washing machine on a temperature of 30 degrees or less and allow to air dry
- Store in your box provided by Dearson

BATTERY

- Wipe over with a damp cloth
- Dry thoroughly with a soft dry cloth
- Charge after use and remove from charger once fully charged
- Store in the box provided by Dearson

CASE

- Wipe over with a damp cloth
- Dry thoroughly with a soft dry cloth

CONTACT:

+44 121 773 4100

ENQUIRIES@DEARSONLTD.COM



DEARSON®
Veterinary Products